A.72525  
SMALL

A.75025  
MEDIUM

A.75040  
LARGE

# Spongy

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Sterile sponge for wound cleansing

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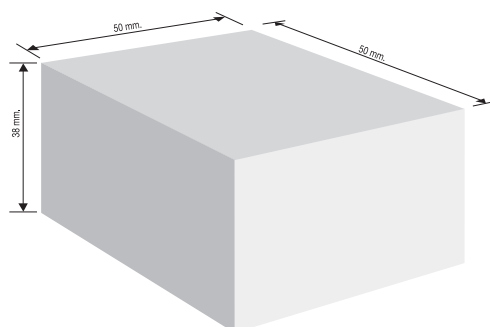
Managing wounds of various surface areas and depths, consequent on surgical operations, is onerous for both the patient and the healthcare system. Frequent medication is necessary for long periods of time, involving a considerable investment of time by medical and paramedical staff. The use of a disposable, sterile, biocompatible sponge in polyurethane wadding with a detergent effect constitutes substantial progress in medicating systems compared with gauze packing. Applied inside the wound, it is soaked with exudates after a few hours. Changing Spongy every 6-8 hours allows for frequent evacuation of serum and fibrous or purulent secretions, which will facilitate the granulation of the repair tissue and thus promote healing. It is indicated for use after perfect haemostasis has been achieved by packing the wounds in the operating theatre, replaced it on the first and, if necessary, on the second day after the operation.


Spongy allows the patient to medicate him or herself quickly, and in a familiar environment.

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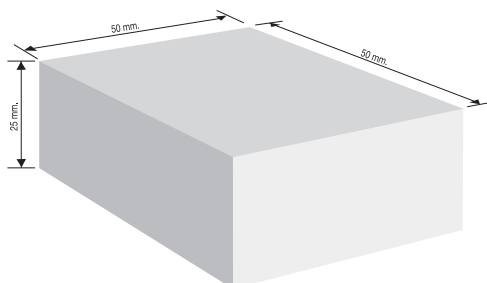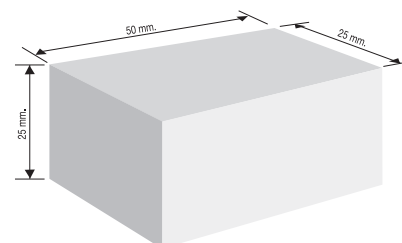
### Instructions for use

The first time the physician should show the patient how to medicate his or her wound with **Spongy**.

Trim **Spongy** into a shape as close as possible to that of the wound with scissors that have been thoroughly disinfected with hydrogen peroxide, **moisten with physiological solution, squeeze dry and place inside the wound**, applying light pressure to ensure the edges of the sponge touch the walls of the wound. A gauze taped to the skin will keep the sponge in the correct position until it is to be changed, 6-8 hours later, or as advised by the physician. In subsequent cleansing operations the patient can shape the other sponges in a similar way – the sizes will gradually reduce – until the wound is completely healed.

Every 7/15 days physician can evaluate whether or not the medication is correctly performed.

Using **Spongy** instead of traditional medication with packing offers the following advantages:

- the sponge may be changed several times a day, with better wound cleansing;
- patients may medicate the wound themselves, with a considerable advantage in organisational terms;
- frequent wound cleansing promotes faster scar formation and healing.

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**LUJ** from Sapi Med is transparent, water-soluble and sterile, and provides continuous and prolonged lubrication. It is compatible with the mucous membranes, flexible endoscopic instruments and surgical instruments. The line offers a wide range of packages and formats to satisfy the needs of both specialist and patient.

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500ml dispenser in recyclable plastic. **Non sterile** jelly for rectal and gynaecological examinations, and to lubricate flexible endoscopic instruments.

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250ml tube. **Non sterile** jelly for rectal and gynaecological examinations, and to lubricate flexible endoscopic instruments.

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10ml single use sachet **sterile** jelly for catheter insertion, rectal and gynaecological examinations, and to lubricate flexible endoscopic instruments. Box of 20 sachets.

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**Package for hospitals use only:**  
A.3510.100: box containing 100 sachets

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**Sterilised:** with beta radiation